

## ChloroPrep® 26 ml cutaneous solution

2% Chlorhexidine Gluconate w/v and 70% Isopropyl Alcohol v/v<sup>1,2</sup>  
Single patient use, sterile, non-touch, skin antisepsis system

### For procedures such as:

Surgical site antisepsis, implantable device placements,  
prosthetic device placements, femoral graft sites

Coverage area 50 cm x 50 cm



### Pinch

Hold the applicator as shown, being careful not to touch the sponge.

Pinch the lever to release the solution. You will hear a 'pop' as the ampoules break.



### Apply

Starting at the incision site, gently press the applicator against the skin and apply using repeated up and down, back and forth strokes, before working outward to the periphery.

For **dry surgical sites** (e.g. abdomen or arm) apply for at least 30 seconds.

For **moist surgical sites** (e.g. inguinal fold or axilla) apply for at least 2 minutes.



### Dry

Leave the area to **air dry completely** before applying sterile drapes.

For **dry sites** allow approximately 2 minutes.

For **moist sites** allow approximately 3 minutes.

Discard the applicator after a single use.

**IMPORTANT SAFETY POINT:** Do not drape or use ignition source until solution is **completely dry**.