

## ChloroPrep® FREPP 1.5 ml cutaneous solution

2% Chlorhexidine Gluconate w/v and 70% Isopropyl Alcohol v/v<sup>1,2</sup>  
Single patient use, sterile, non-touch, skin antisepsis system

### For procedures such as:

Blood culture collections, peripheral canulation, AV fistula  
and 'portacath' access, simple biopsies and minor procedures

Coverage area 10 cm x 13 cm



### Pinch

Hold the applicator as shown, being careful not to touch the sponge.

Pinch the wings together. You will hear a 'pop' as the ampoule breaks.



### Apply

Gently press the applicator against the skin and apply the antiseptic using repeated up and down, back and forth strokes for about 30 seconds.



### Dry

Leave for approximately 30 seconds, allowing the area to air dry **completely** before proceeding.

Discard the applicator after a single use.