

ChloroPrep® SEPP .67 ml cutaneous solution

2% Chlorhexidine Gluconate w/v and 70% Isopropyl Alcohol v/v^{1,2}

Single patient use, sterile, non-touch, skin antiseptic system

For procedures such as:

Peripheral canulation, surgical pin site cleansing,
subcutaneous device insertions

Coverage area 5 cm x 8 cm



Pinch



Hold the applicator as shown, being careful not to touch the tip.

Pinch the sides of the applicator until you feel the inner ampoule break.



Apply



Gently press the applicator against the skin and apply the antiseptic using repeated up and down, back and forth strokes for about 30 seconds.



Dry



Leave for approximately 30 seconds, allowing the area to air dry **completely** before proceeding.

Discard the applicator after a single use.